




# neodisher® MediKlar Dental

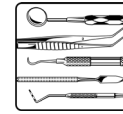

## Rinse aid for the automated reprocessing of dental instruments



### Liquid concentrate

#### Fields of application:

- Rinsing during automated reprocessing of dental instruments incl. drive units/handpieces as well as dental utensils such as trays, mouth rinsing beakers and kidney dishes

#### Performance spectrum:

- Improves the wetting of surfaces made of stainless steel and plastic
- Accelerates the drying process in automated reprocessing
- When neodisher MediKlar Dental is used as intended, the biocompatibility of the treated instruments is not affected. (Biological evaluation of medical devices according to EN ISO 10993-1)

#### Special properties:

- Reduces the drying time significantly
- Neutralises carried over alkaline residues and alkaline water

#### Application and dosage:

neodisher MediKlar Dental is used in dental washer disinfectors. neodisher MediKlar Dental is dosed automatically via integrated dosing devices in the final rinse water and is 0.2 - 0.4 ml/l.

The dosing devices must be suitable for dosing neodisher MediKlar Dental.

#### Notes on application:

- For professional use only.
- Do not mix with other products.
- Rinse out dosing system including suction hose with water before changing product.

- The instructions given by the manufacturer of the washer disinfectant are to be observed.
- Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated processes.
- Please observe the reprocessing recommendations of the medical device manufacturers according to the requirements of the DIN EN ISO 17664.

#### Technical data:

pH-value	4.0 - 3.7 (0.2 - 0.4 ml/l, determined in deionised water, 20 °C)
Viscosity	< 50 mPa·s (concentrate, 20 °C)
Density	approx. 1.1 g/cm <sup>3</sup> (20 °C)

#### Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:  
 < 5 % phosphonates  
 15 - 30 % non-ionic surfactants  
 also preservatives (methylchloro-isothiazolinone/methylisothiazolinone)

CE-mark:

neodisher MediKlar Dental complies with European guidelines for medical devices.


If a serious incident occurs with the product, report it to the manufacturer and the relevant national authority.

#### Storage information:

The product is sensitive to frost. Always store at a temperature between 0 °C and 30 °C. Usable for 3 years when stored as recommended. For



# neodisher<sup>®</sup> MediKlar Dental

expiry date refer to the stamp mark on the label behind the hourglass symbol .

## Hazard and precautionary statements:

For further safety information see Safety Data Sheets. These are available e.g. at [www.drweigert.com](http://www.drweigert.com) under the category "Service/Downloads".

Dispose only when container is empty and closed.  
For disposal of product residues, refer to the Safety Data Sheet.

MB 4776/3-1  
Date of issue: 05/2020

---

With the above information, to our current knowledge we describe our product regarding safety necessities, but we do not involve any quality description or promise certain properties.