



neoform[®] D plus



Surface disinfectant for medical devices



Liquid concentrate

Fields of application:

- For the cleaning and disinfection of bed-frames and other washable medical devices in hospitals, doctor's practices and nursing homes.
- Bactericidal and yeasticidal activity. Confirmed by certification.

	Under dirty conditions (pre-cleaned and not pre-cleaned surfaces) 20 °C
Bactericidal / yeasticidal	30 ml/l (3.0 %), 15 min 20 ml/l (2.0 %), 30 min 15 ml/l (1.5 %), 1h 5 ml/l (0.5 %), 4 h

Special properties:

- Good material compatibility.
- Low odour.
- Free of formaldehyde and phosphates.

Application and Dosage:

neoform D plus is used for the combined surface cleaning and disinfection at room temperature via wiping or mechanical cleaning equipment such as rotating brushes or spray cleaners.

Depending on the desired activity dilute neoform D plus according to the below mentioned concentration. Wipe the surface to be cleaned and disinfected with the working solution and adhere to the contact time. After the expiry of the contact time rinse the surface thoroughly with water of drinking quality.

General instructions for use:

- For professional use only.
- It is generally recommended to wear gloves when using disinfectants manually.
- For economical and controlled dosage manual dosing and application ancillaries are available, e.g. mixing equipment or dosing accessories. Please contact us.
- The neoform D plus working solution has to be rinsed off with water (preferably deionised water).
- Do not mix with other products.



neiform[®] D plus

Expert reports / listings:

The disinfecting activity has been confirmed by certification. Expert reports are available on request.

Technical data:

pH-range	5.4 – 4.7 (5 - 30 ml/l, determined in deionised water, 20 °C)
Viscosity	< 50 mPa s
Density	1,05 g/cm ³ (20 °C)

Ingredients:

Ingredients according to Regulation (EC) No 648/2004 on detergents:


< 5 % NTA, 5 - 15 % non-ionic surfactants
also perfumes
actives substances in 100 g:
9.1 g didecyldimethylammoniumchloride
8.0 g glyoxal

CE-marks: 0297

neiform D plus conforms to the requirements of the European directive 93/42/EC, Annex I concerning medical devices.

Storage information:

Always store at a temperature range between 0 and 30 °C.

Usable for 3 years when stored as recommended.
For expiry date refer to the stamp mark on the label behind the hour glass symbol .

Hazard warnings and safety advice:

Classification and labelling according to the Dangerous Preparation Directive 1999/45/EC for the concentrate as delivered.

Hazard symbol Xn - harmful,
Dangerous for the environment
contains: glyoxal (5-10 %),
didecyldimethylammoniumchloride

Risk phrases:

- R 38 - Irritating to skin.
- R 41 - Risk of serious damage to eyes.
- R 43 - May cause sensitization by skin contact.
- R 50 - Very toxic to aquatic organisms.
- R 68 - Possible risks of irreversible effects.

Safety phrases:

- S 26 - In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- S 36/37/39 - Wear suitable protective clothing, gloves and eye/face protection.
- S 46 - If swallowed, seek medical advice immediately and show this container or label.
- S 61 - Avoid release to the environment. Refer to special instructions/safety data sheets.

If applied according to the instructions for use the product is safe in the sense of the appropriate guidelines upon medical devices.

Dispose only when container is empty and closed. For disposal of product residues, refer to Material Safety Data Sheet.

For further safety information see EC-Safety data sheets. These are available at www.drweigert.de under the category „Service“.

MB 3003/3-1
08/2010

Die Angaben dieses Merkblattes beruhen auf unseren derzeitigen Erkenntnissen und Erfahrungen. Sie befreien den Verwender nicht von eigenen Prüfungen und Erfahrungen. Eine rechtlich verbindliche Zusicherung bestimmter Eigenschaften kann hieraus nicht abgeleitet werden.

With the above information, to our current knowledge we describe our product regarding safety necessities, but we do not involve any quality description or promise certain properties.



DR. WEIGERT

Chemische Fabrik Dr. Weigert GmbH & Co. KG
Mühlenhagen 85 • D-20539 Hamburg

Tel.: +49-40-7 89 60-0
Fax: +49-40-7 89 60-120

info@drweigert.de
www.drweigert.de