



neoform[®] wipes RTF

ready to fill



Disposable fleece wipe dispensing system for use with detergents and disinfectants

Plastic bucket with fleece wipe roll

Fields of application:

- Cleaning and disinfection of surfaces, appliances, medical devices and medical equipment in combination with suitable Dr. Weigert detergents and disinfectants in hospitals, old people's homes/nursing homes, in chemical and pharmaceutical areas, laboratories as well as in food processing companies and professional kitchens
- Also suitable for high-risk areas such as neonatology or haematological oncology in accordance with the VAH¹ recommendation²

Performance spectrum:

- Multi-purpose, compatible with a broad range of Dr. Weigert surface disinfectants, proved using the 4-fields-test (EN 16615)
- Disinfecting action of the soaked fleece wipes during the maximum shelf life of 28 days proved using the 4-fields-test (EN 16615)
- Stable dispensing system with integrated wipe roll and stable handle enable simple, safe and user-friendly handling
- High hygiene safety thanks to re-sealable system prevents drying and contamination
- Meets – in combination with the below-mentioned surface disinfectants – the requirements of the “recommendation for controlling critical points when using wipe dispensing systems in a pre-soaking system for surface disinfection” of the VAH¹

Special properties:

- Absorbent, lint-free and tearproof disposable fleece wipes for optimum results
- Specially designed drop-shaped opening enables easy separation of the disposable fleece wipe and therefore allows easy re-sealing

- Disposable fleece wipe dispenser system – saves time and cost-intensive reprocessing
- Tamper-evident closure
- Stackable and space-saving

Application and dosage:

1. Before preparing the ready-to-use disposable wipe dispensing system disinfect hands and wear protective gloves.
2. Removal of the wipe dispenser lid by opening the tamper-evident closure. Then slowly fill centrally at least 2.25 L of the ready-to-use, alcoholic surface disinfectant resp. at least 2.5 L of the application solution of the Dr. Weigert product in the required concentration in accordance with the application recommendation.
3. Pull the first fleece wipe from the middle of the wipe roll through the opening of the dispensing lid, pull to the tapered end of the opening, tear off and discard.
4. Place the lid on the bucket and tightly close the opening. Fill in the preparation information on the wipe dispenser system (product name, batch, date of preparation, concentration, use-by date). After 30 min the fleece wipes are completely soaked and ready to use.
5. Wipe the surfaces to be cleaned or disinfected carefully with the soaked neoform wipes. For safe disinfection make sure the surfaces are completely wetted and observe the product specific contact time. Then discard the fleece wipe.
6. Keep the wipe dispenser system tightly closed after taking out the neoform wipes.
7. Disposal system: After depletion or after expiry of the shelf life, properly recycle wipes, any product residues and dispenser system.

The soaked neoform wipes can be used for up to 28 days if the dispenser system is properly closed.



neoform[®] wipes RTF



Compatibility:

The neoform wipes RTF disposable wipe dispensing system can be used with the following Dr. Weigert surface disinfectants:

- neoform Classic
- neoform MED FF
- neoform Rapid
- neoform K plus
- neoform K sprint
- neoform K dis

Please observe the application recommendations given in the product data sheet of the respective surface disinfectant. These are available under the category "Service/Downloads" at www.drweigert.com.

Notes on application:

- For professional use only.
- Not suitable for baby care.
- Not suitable for skin disinfection.
- Do not dispose in sanitary facilities.
- Do not use dried wipes.

Technical data:

Material	100 % PET
Wipes per roll	115
Weight per roll	425 g
Size of fleece wipe	30 x 25 cm

Storage information:

Protect from dust, humidity and direct sunlight. Store closed container at room temperature. It is recommended to limit the storage time of the closed container to a maximum of 5 years.

Hazard and precautionary statements:

Dispose only when container is empty and closed. For disposal of product residues, refer to the product-specific Safety Data Sheet. This is available at www.drweigert.com under the category "Service/Downloads".

MB 4213/2-3

Last updated: 07/2022

¹ Verbund für Angewandte Hygiene [Association of Applied Hygiene]

² Mitteilung der Desinfektionsmittel-Kommission im VAH unter Mitwirkung der „4+4-Arbeitsgruppe“: Zur Verwendung von Tuchspendersystemen in Bereichen mit besonderem Infektionsrisiko, Hyg Med 2014

[Notification of the Disinfectants Commission in the VAH with the participation of the "4 + 4 Working Group" On using wipes dispensing systems in areas of increased risk of infection, Hyg Med 2014]